

A.M.I.® Easy Instrument System

Product Group Laparoscopy | Issue 04/2012



> Save on costs. Save on cleaning time. Save your back!








- Reusable handles with or without ratchet mechanism and integrated, manually-operated cautery function
- Instruments allow more freedom of movement for the operator = less back pain
- Can be fully disassembled for quick, thorough cleaning
- Combine with various disposable and reusable tips for cutting, grasping, dissecting and holding tissue

A.M.I.® Easy Instrument System

A.M.I. Easy Instrument Premium and EasyAct

Reusable laparoscopic instruments: Quality and cost-efficiency in one.

- The instruments can be fully disassembled for fast thorough cleaning. That means the highest safety for both user and patient.
- Ergonomic handles allow a comfortable, sure grip on the instrument.
- Colour coding ensures you'll reach for the right instrument every time.
- The fully-rotating shaft facilitates handling and allows universal application
- More control and flexibility for the surgeon

Order Code	A.M.I. Easy Instrument System	Technical Details
ESI1110 	A.M.I. Easy Instrument Premium Reusable instrument for use with the A.M.I. Easy Instrument Premium Tips, in particular the Shear Tips ESI1131, ESI1151, ESI1161, ESI1120 and ESI1140	5 mm shaft Length: 330 mm Rotation: 360° 1 instrument, fully detachable Delivered non-sterile, steam autoclavable
ECI1010 	A.M.I. EasyClinch Instrument Premium Reusable instrument with ratchet. For use with the A.M.I. Easy Instrument Premium Tips, in particular the Clinch and EasyFinger Tips ECI1031, ECI1041 and EFI3001	5 mm shaft Length: 330 mm Rotation: 360° 1 instrument, fully detachable Delivered non-sterile, steam autoclavable
EAS1020 	EasyAct Reusable instrument with integrated, manually-operated cautery function. For use with the A.M.I. Easy Instrument Premium Tips, in particular the Shear Tips ESI1131, ESI1151, ESI1161, ESI1120, ESI1140 and Hook EDI1051	5 mm shaft Length: 330 mm Rotation: 360° 1 instrument, fully detachable Delivered non-sterile, steam autoclavable
ECA1020 	A.M.I. HF-Cable Martin HF-cable for connecting EasyAct with a high-frequency generator from KLS Martin. For monopolar use	Length: 3 m 1 cable Delivered non-sterile, steam autoclavable
ECA1030 	A.M.I. HF-Cable Erbe HF-cable for connecting EasyAct with a high-frequency generator from Erbe. For monopolar use	Length: 3 m 1 cable Delivered non-sterile, steam autoclavable
ECA1040 	A.M.I. HF-Cable ValleyLab HF-cable for connecting EasyAct with a high-frequency generator from ValleyLab. For monopolar use	Length: 3 m 1 cable Delivered non-sterile, steam autoclavable
ECA1110 	A.M.I. HF-Cable Adapter Adapter for connecting A.M.I. HF-Cables to instruments other than EasyAct (including instruments from other manufacturers)	1 adapter Delivered non-sterile, steam autoclavable

A.M.I.® Easy Instrument System

More room to move with the EasyAct



To activate high-frequency current, simply press the trigger on the instrument's handle. There's no longer any need to keep your foot on a pedal!

- Safer stance
- No additional pedals under the operating table
- More freedom of movement, particularly during long procedures
- Less strain on the operator's back

A.M.I. EasyShears - Perfect cutting, simple cleaning, highly cost-efficient




- Sterile disposable shear tips provide optimum cutting performance combined with a high level of safety
- Reusable shear tips are cost-efficient and environment-friendly
- Extra-long bariatric tips for use on overweight patients

A.M.I. EasyClinch - The safe, atraumatic grasper




- The very wide jaw angle allows safe grasping of all specimen, even the largest gallstones
- Keeps a firm hold on tissue without penetrating
- Extra-long clinch tips for use on overweight patients

A.M.I. EasyFinger - Flexible finger tips for positioning gastric banding implants and carrying out fundoplication surgery

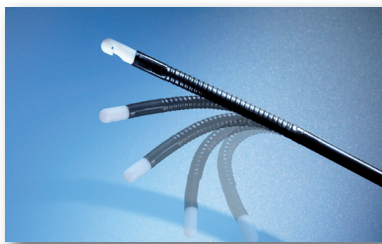

- Flexible finger with a bending angle of 110°; can be locked in position
- Special geometry for hooking on to and keeping hold of loops
- Extra-long shaft covered in PTFE plastic for use on obese patients

A.M.I. EasyHook - The hot tip for precise preparation

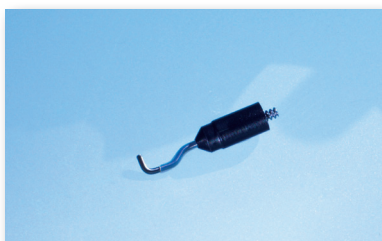

- Disposable, insulated hook with electrocautery capability for dissection and coagulation in just one instrument
- Fast, safe, controlled preparation with minimal waste



A.M.I.® Easy Instrument System

All tips may be used interchangeably with any of the three instrument handles - A.M.I. Easy Instrument Premium, A.M.I. EasyClinch Instrument Premium and EasyAct.

Order Code	A.M.I. EasyShears	Technical Details
ESI1131	A.M.I. EasyShear Disposable Tips Premium	Curved Metzenbaum blades, 16 mm long
	Disposable shear tips	Delivered sterile 10 tips / box
ESI1151	A.M.I. EasyShear Mini Disposable Tips Premium	Curved Metzenbaum blades, 10 mm long
	Disposable shear tips	Delivered sterile 10 tips / box
ESI1120	A.M.I. EasyShear Reusable Tips Premium	Curved Metzenbaum blades, 16 mm long
	Reusable shear tips	Delivered non-sterile, steam autoclavable 5 tips / box
ESI1140	A.M.I. EasyShear Mini Reusable Tips Premium	Curved Metzenbaum blades, 10 mm long
	Reusable shear tips	Delivered non-sterile, steam autoclavable 5 tips / box
ESI1161	A.M.I. EasyShear Bariatric Disposable Tips	Curved Metzenbaum blades, 16 mm long Tips 28 mm longer than ESI1131
	Disposable shear tips for use on obese patients or during any procedure requiring a longer instrument	Delivered sterile 10 tips / box
Order Code	A.M.I. EasyClinch	Technical Details
ECI1031	A.M.I. EasyClinch Disposable Tips Premium	
	Disposable clinch tips	Delivered sterile 10 tips / box
ECI1041	A.M.I. EasyClinch Bariatric Disposable Tips	Tips 27 mm longer than ECI1031
	Disposable shear tips for use on obese patients or during any procedure requiring a longer instrument	Delivered sterile 10 tips / box
Order Code	A.M.I. EasyFinger Disposable	Technical Details
EFI3001	A.M.I. EasyFinger Disposable	Length: 156 mm
	Disposable finger tip for positioning gastric banding implants and carrying out fundoplication surgery	Delivered sterile 10 tips / box
Order Code	A.M.I. EasyHook	Technical Details
EDI1051	A.M.I. EasyHook	
	Electrocautery hook for tissue dissection with coagulation	Delivered sterile 10 tips / box

A.M.I. Headquarters:

A.M.I. GmbH
Im Letten 1
6800 Feldkirch . Austria
t +43 5522 90505-0
f +43 5522 90505-4006
e info@ami.at
www.ami.at